


COVID-19 SARS-CoV-2 Antigen Kit (Saliva)

For Self-testing

**Rapid
Innovative
Comfortable**

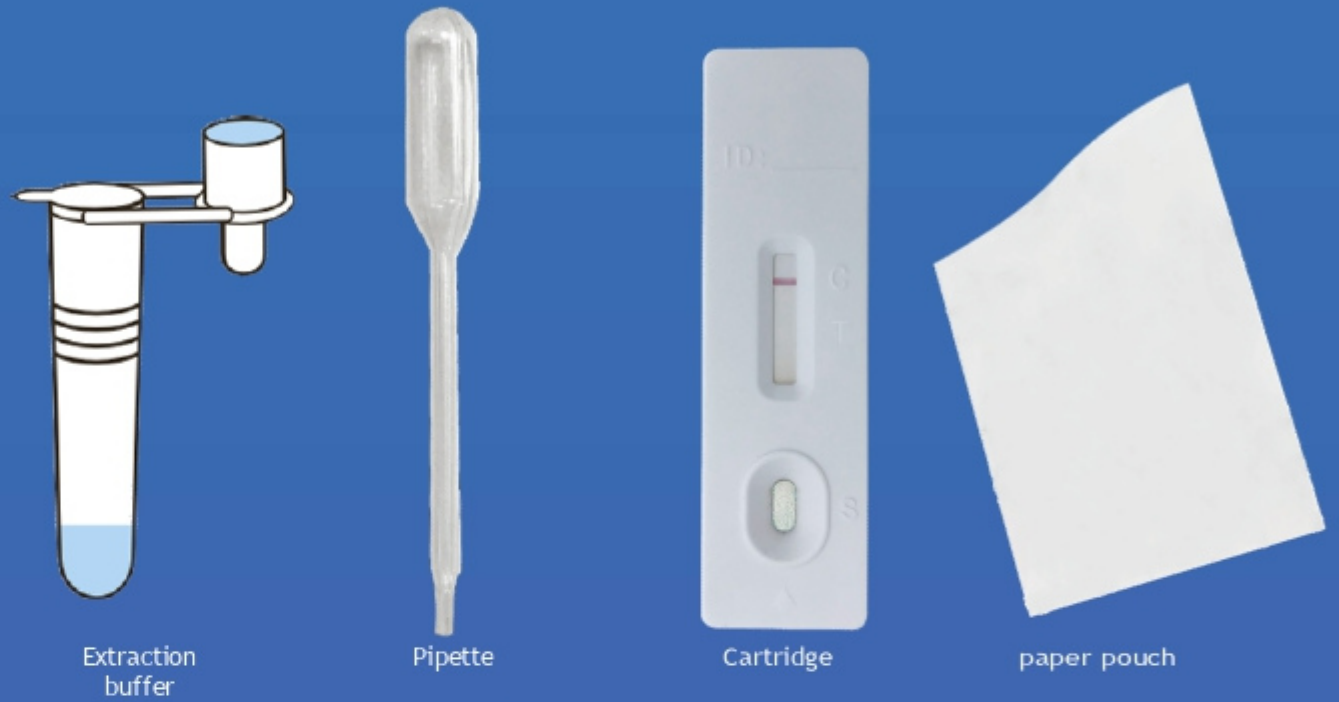


Kit Components

Test procedure

- 1 Open the paper pouch and directly spit saliva into it
- 2 Use a disposable pipette to absorb the saliva specimen
- 3 Peel off the sealing film on the pre-filled Extraction Tube.
- 4 transfer the saliva specimen into the extraction tube
- 5 Attach and tighten the Dropper Lid
- 6 Leave the swab for 1 minute.
- 7 Take out the Test Cartridge. Add 3 drops into sample well. After the 12-minute incubation, read the results.

Interpretation of Results

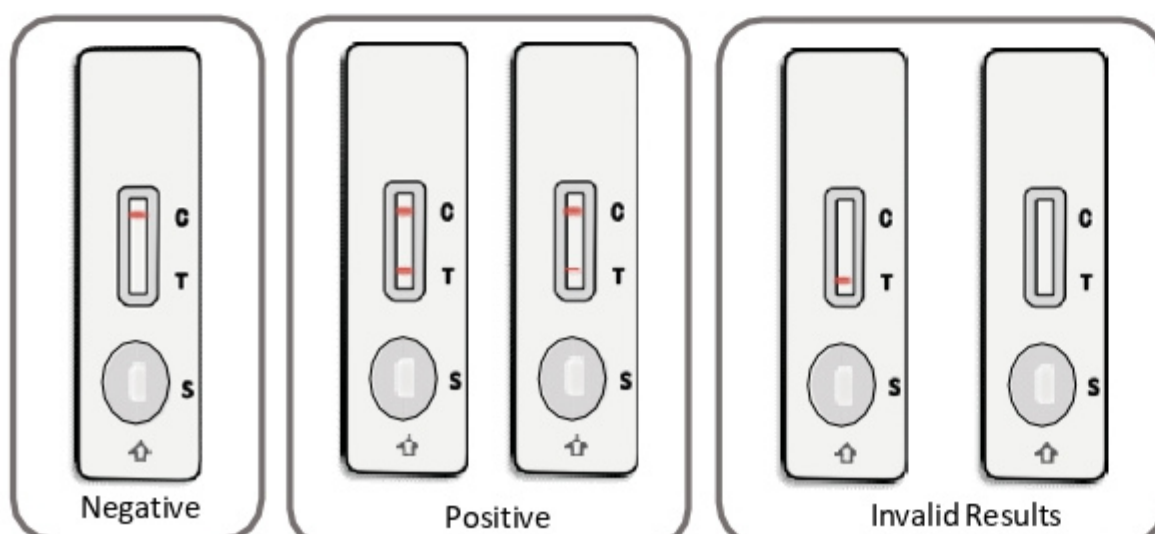

ORDER NOW

012-684 2078



sales@vigagemilang.com.my



B-G-8, RAWANG IDAMAN JALAN IDAMAN 1,
TAMAN RAWANG IDAMAN, 48000 RAWANG,
SELANGOR, MALAYSIA

